Snelgids

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Add a new patient with the minimum number of details. You can add more details later under 'your patient'.

Patient profile

surname (*): Peeters
first name (*): Jan
date of birth: year = 2016 month = 04 day = 02

1

2

Patient "Peeters Jan" is successfully added.

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patient sessions
your patient sessions

Supported actions

Add new patient sessions and update or delete formerly defined patient sessions

your patient sessions

Select a template or session:

- your session templates
- progress & results
- end active sessions
- assign raters
- back to main page

Session properties

date of creation (yyyy-mm-dd):
description (maximum 32 characters):
selected patient:
selected exercises:
add an exercise:
number of times to do (default 1):

1

---- templates for quickly creating new sessions ----

voorbeld
SKObasis, SIG, vzw, ga
SKOodd, SIG, vzw, ga
SKOinco, SIG, vzw, ga
---- already defined sessions ----

2021-05-21, Caro, Lientje
2021-05-09, Caro, Lientje

Supported actions

Add new patient sessions and update or delete formerly defined patient sessions

your patient sessions

Select a template or session:

- your session templates
- progress & results
- end active sessions
- assign raters
- back to main page

Session properties

date of creation (yyyy-mm-dd):
description (maximum 32 characters):
selected patient:
selected exercises:
add an exercise:
number of times to do (default 1):

2

add/update delete clear form

2021-09-27
inconsistenciestest
Peeters, Jan

SKO.inco.benoem(SIG,vzw,ga) SKO.inco.zegna(SIG,vzw,ga)

1

SKO - © Sig vzw - www.SKOTest.be

VOORAF

experiments
try exercise

test microphone Exercise to select:

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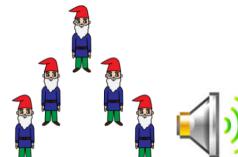
SKO - © Sig vzw - www.SKOTest.be

3. Neem de module het SKO-Inconsistentie af

experiments **record patient session**

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SKO – © Sig vzw – www.SKOTest.be

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experiments **make perceptual ratings** **select a session**

Supported actions Select the session that you want to rate perceptually.

Patient session:

- 2021-05-29, Caro, Lienje, 0/2 ratings completed
- 2020-08-29, Ieon, Jef, 7/7 ratings completed
- 2020-02-07, Caro, Lienje, 1/1 ratings completed
- 2019-10-21, More, Thomas, 0/7 ratings completed
- 2019-01-08, Pek, Pietje, 7/7 ratings completed
- 2017-11-14, M29190, YVA, 1/1 ratings completed

EXERCISE UNDER EVALUATION: SKO.inco.benoem

SELECT THE SPEECH TASK RECORDING YOU WANT TO LISTEN TO

SELECT THE TRANSCRIPT OF THE SELECTED SPEECH TASK RECORDING

stop rating

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SKO – © Sig vzw – www.SKOTest.be

5. Haal de resultaten op

patient sessions		progress & results	
your patient sessions	patient name	nr of sessions defined / completed	
your session templates	But Jeanne	1 / 1	sessions
progress & results	But Jerom	1 / 1	sessions
end active sessions	Caro Lientje	3 / 3	sessions
assign raters	DDK Joris	7 / 7	sessions
back to main page	john F		
	leon jef		
	M29190 YVA		
	More Thomas	1 / 1	sessions
	Name Louise	1 / 0	sessions
	Peeters jan	2 / 2	sessions
	Pek Pietje	3 / 3	sessions
	Slagers Georges-Louis	2 / 2	sessions

Supported actions Survey of the sessions that were created for patient Peeters jan. Click on a **more** button (if any) to consult more details about the self-evaluations, the automatic particular session.

one level back	patient session + description	session cycles (due / done)	session analyses (due / done)	session ratings (open / busy / done)
your patient sessions	Peeters jan (inconsistentietest) 2021-09-27 (diadochokinese)	1 / 1	0 / 0	0 / 0 / 1 more
progress & results		3 / 3	6 / 6 more	0 / 0
end active sessions				

Supported actions Perceptual ratings delivered for a particular session. To print the report: put the cursor in this window, right click on the mouse and select the PRINT option.

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