
# Snelgids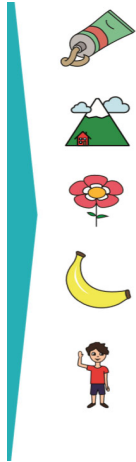

© Sig vzw  
versie september 2022  
Sig vzw  
Pachthofstraat 1, 9308 Gijzegem  
[info@sig-net.be](mailto:info@sig-net.be)

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add new patient

What you can do

You can add a new patient by entering a minimum number of details (only those marked by a star are obligatory) and by clicking on 'ADD'. If you need more details go to 'your active patients' to get a more elaborate form.

- add new patient
- your active patients
- your archived patients
- your patient groups
- mail to patient
- customize emoticons

Patient profile  
last name (\*):  
first name (\*):  
date of birth:

add patient clear form

Peeters  
Jan  
year = 2016 month = 04 day = 02

1

Patient "Peeters Jan" is successfully added.

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define/check sessions

select/change patient

Select the desired patient: Peeters,Jan

define/check sessions

You can perform actions on the sessions of this patient (see menu on the left) or select another patient.

1

Select session to start from:  
Select session prototype to start from:  
PATIENT SESSION FORM  
name of session (yyyy-mm-dd):  
description (max. 32 characters):  
selected patient:

DJA  
SKObasis[SIG.vzw.ga]  
SKOdlk[SIG.vzw.ga]  
SKOimco[SIG.vzw.ga]

Peeters,Jan

2

3

PATIENT SESSION FORM  
name of session (yyyy-mm-dd): 2022-07-23  
description (max. 32 characters): diadochokinese  
selected patient: Peeters, Jan  
exercise sequence:  
add an exercise:  
number of times to do (default 1): 3

add update delete clear form

Session "2022-07-23" was successfully added.

SKO - © Sig vzw - www.SKOtest.be

3. Neem de module DDK af: stap 3= neem de test af (3x potteke en papa)

experiments

record patient session

select patient session

Select patient with sessions:

Peeters, Jan

record patient session

Select a session of that patient:

2022-07-21 (inconsistentietest)

listen to responses

You can perform actions on the selected session (see menu on the left) or select another session.

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