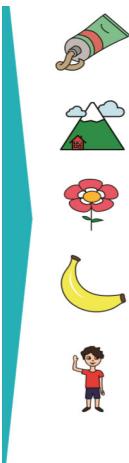
Snelgids

© Sig vzw
versie september 2022

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patients

What you can do 

- add new patient**
- your active patients
- your archived patients
- your patient groups
- mail to patient
- customize emoticons

patient sessions

experiments

main page

add new patient

You can add a new patient by entering a minimum number of details (only those marked by a star are obligatory) and by clicking on 'ADD'. If you need more details go to 'your active patients' to get a more elaborate form.

Patient profile

last name (*):

first name (*):

date of birth:

year = month = day =

1

Patient "Peeters Jan" is successfully added.

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1

define/check sessions

You can perform actions on the sessions of this patient (see menu on the left) or select another patient.

Select session to start from:

Select session prototype to start from:

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SKO - © Sig vzw - www.SKOTest.be

3. Neem de module Inconsistentie af: stap 3= neem de test af

experiments **record patient session**

select patient session
record patient session
listen to responses

Select patient with sessions:
Select a session of that patient:
Peeters.Jan
2022-07-21 (inconsistentietest)

You can perform actions on the selected session (see menu on the left) or select another session.

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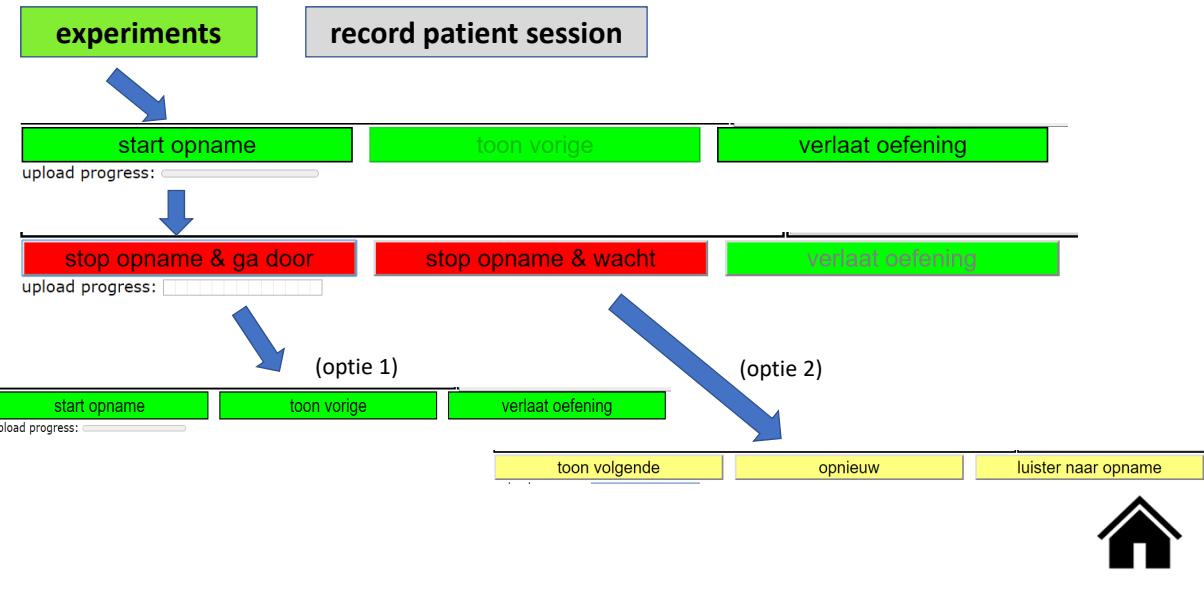
make perceptual ratings

EXERCISE UNDER EVALUATION: SKO.inco.benoem

SELECT THE SPEECH TASK RECORDING YOU WANT TO LISTEN TO

01 02 03 04 05

1

SELECT THE TRANSCRIPT OF THE SELECTED SPEECH TASK RECORDING

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main page

Caro Lientje (session: 2022-07-07 inconsistentietest)

Automatic analysis results for this session

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